

# BIONASE

**Clinically Tested Drug-Free**

**HAY FEVER TREATMENT**

- DRUG-FREE TREATMENT
- ALLERGIC RHINITIS RELIEF
- SIMPLE & SAFE TO USE



**Intelligent**  
health systems

[www.bionase.com.au](http://www.bionase.com.au)



**Intelligent**  
health systems

# BIONASE

**Clinically Tested Drug-Free**

**HAY FEVER TREATMENT**

## What is Bionase®?

Bionase® is a revolutionary allergy treatment that works by phototherapy, to treat hay fever and nasal allergies without the use of medicine.

## What is phototherapy?

Bionase® phototherapy is a recently developed method for the treatment of allergic reactions. It uses a visible red light source at a single wavelength. The red light delivers the required effect of suppressing the allergic symptoms and reduction in the sensitivity of the nasal mucous membrane. Up until now these diodes have been prohibitively expensive. It was only possible to carry out phototherapy at specialised centres.

However, the development of Bionase now means that it is possible for allergy sufferers themselves to carry out this highly effective, 4½ minutes long treatment.

## How the Bionase® works

The specific red light emitted by the Bionase light source protects the mucous membrane of the nose against allergic agents.

- **Frequent use of the red light, ensures that the immune cell receptors in the nose deliver a delayed reaction to pollen**
- **As a result of the red light therapy, the walls of the mast cells become less sensitive and penetrable, which results in less histamine release.**
- **The combined action of the immune cells is influenced locally, reducing the reaction to allergic agents.**

## How to use the Bionase®

Each Treatment lasts for 4½ minutes during which time a red light is emitted by a source. By pressing the start button on the Bionase®, the light source will be switched on and after 4½ minutes it will turn itself off, so that treatment times are exactly the same each time. To start with, treatment should be carried out 3 times per day. For extreme allergic symptoms, treatment can be repeated more often without any side effects. As soon as the symptoms have abated, the number of the treatments can be reduced to 2 per day. If the results improve, you can cut this down to 1 per day.

It is important to continue treatment for at least 1 to 2 weeks in order to maintain the results. It is best to perform the treatment in the morning and repeat it again at night. If you need 3 treatments a day, just take Bionase with you. If you know which type of pollen causes your allergy, consult the pollen calendar and start the Bionase® treatment 1 week beforehand so that the allergic symptoms can be prevented or eliminated.

## The Proven Success of the Bionase®

The Bionase® has undergone clinical trials and has resulted in a significant reduction of allergic symptoms in most patients. These trials showed that the mucous membrane became less sensitive and a reduction in the allergic symptoms occurred within two weeks. In most cases, the Bionase® will provide rapid relief. Some symptoms may abate or even disappear within the space of a few hours. In the other cases this will take a few days. The main benefit of the Bionase® phototherapy treatment is that it is highly localised and does not affect other parts of the body as is the case with a great many medicines. Furthermore, the Bionase® provides long term relief with the alleviation and even disappearance of the allergic symptoms.

*Suitable for 4 years and older. Consult your doctor before use if pregnant.*



**Intelligent**  
health systems

Manufactured by  
Syro Technologies LTD. for  
**INTELLIGENT HEALTH SYSTEMS**  
Factory 2, 8-12 Coora Road,  
South Oakleigh Victoria 3166  
AUSTRALIA

[www.bionase.com.au](http://www.bionase.com.au)  
[www.intelligenthealthsystems.com.au](http://www.intelligenthealthsystems.com.au)

